


Urine Monovette® User Guide

1

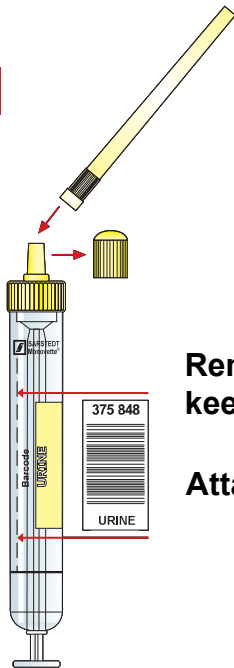

Remove stopper and keep for later use!

Attach plastic straw.

2




To avoid overfilling, insert the plastic straw in to the container and fill the Urine Monovette® up to the bottom of the label as indicated by the red line.

3



To empty the plastic straw hold the Urine Monovette® in an upright position and pull the plunger backwards to the bottom of the tube.

4

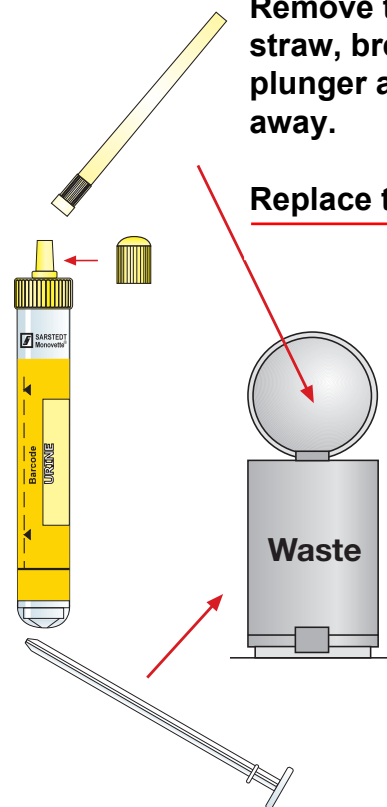

Remove the plastic straw, break off the plunger and throw away.

Replace the stopper.

Technical modifications reserved



SARSTEDT

SARSTEDT Ltd. · 68 Boston Rd, Leicester LE4 1AW · Tel.: 0116 235 9023 · Fax: 0116 236 6099

info.gb@sarstedt.com. www.sarstedt.com